



D. Lux

DialDent

www.diadent.com 1.877.342.3368

CORDLESS LED CURING LIGHT

Unleash the POWER...



OLED PANEL • Unique and intuitive

One-button control



ALUMINUM BODY

• Robust and ergonomic

Easy to disinfect



POWERFUL OUTPUT

• 1600 mW/cm²

• 420 - 490 nm



ADVANCED BATTERY

• 90 minutes fast recharging time

• Battery life indicator




FEATURES

- OLED panel: unique and advanced one-button control
- Cordless, highly portable, and lightweight (only 155 g)
- Unique ergonomic design: provides comfortable grip and easy-to-read OLED panel
- Durable one-piece aluminum design is robust and easy to disinfect
- A high light intensity of 1,600 mW/cm² offers fast and reliable polymerization
- Advanced built-in protection circuit module to protect the unit from over-charging and discharging
- High-performance lithium-ion battery technology:
 - * A new fully charged battery provides 160 x 10-second cures
 - * OLED panel conveniently indicates battery life and a countdown timer for the next needed charge
 - * Battery can be fully recharged in 90 minutes and is replaceable
- 5 versatile curing modes for every indication including turbo, soft start and pulse mode which can reduce the problem of rapid contraction when composite material is being polymerized.

CURING MODES

DiaDenri

5 Max:	Maximum light intensity output for 5 seconds
10 Max:	Maximum light intensity output for 10 seconds
20 Mid:	70% output of maximum intensity for 20 seconds
20 Soft-Start:	An initial 10% output of maximum light intensity that
	gradually increases to full power in 10 seconds,
	followed by 10 seconds output at full power
20 Pulse:	Pulses every 0.15 seconds for 20 seconds at
	maximum light intensity

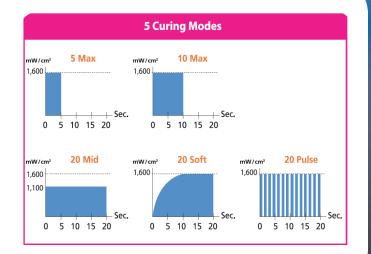

PRODUCT SPECIFICATIONS

Light Wavelength Range: Light Intensity: Charging Time: Operation Time per Full Charge: 10 seconds x 160 times Weight - Hand-piece: Weight - Light Guide: Weight - Charger: Operating Condition: Warranty:

420 ~ 490 nm 1,600 mW/cm² Approx. 90 minutes 125 grams 30 grams 150 grams -10°C ~ 40°C 2 years



ORDERING

#4004-1001	D-Lux Complete Kit	
Contains: D-Lux Har	nd-piece, Charger Base, Light Guide, Battery,	
Plastic Protector, Power Supply and Cord.		
#4004-1110	D-Lux Hand-piece	
#4004-1120	D-Lux Charger Base	
#4004-1130	D-Lux Light Guide (11 x 11 mm)	
#4004-1150	D-Lux Battery Pack	
#4004-1160	D-Lux Plastic Protector	
#4004-1170	D-Lux Power Supply and Cord	

Dia Dent

For more information contact DiaDent at 1.877.DIA.DENT, visit www.diadent.com or call an authorized DiaDent distributor for more information.