





Before treatment

Titanium alloy materials;

P54- Root furcation series








After treatment



Use P54L for scaling of

16 palatal surface.

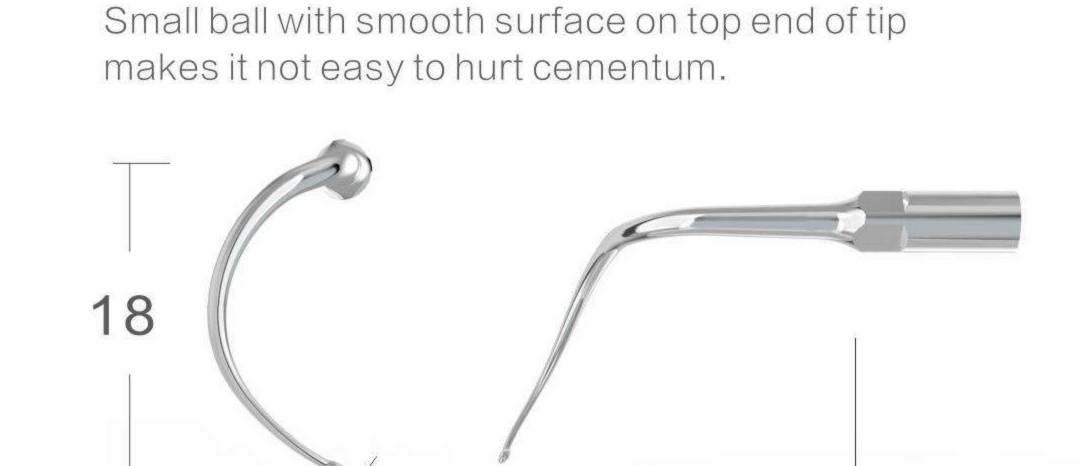

The bending design of P54 makes it fit the tooth surface. And the tip can be deep into the root surface groove, pear-shaped ditch and top of root furcation ridge.



## Root furcation kits






Include one piece of P54L, one piece of P54R and a 2# tips holder.






44~48 lingual root furcation area, mesiocclusion and distal root surface

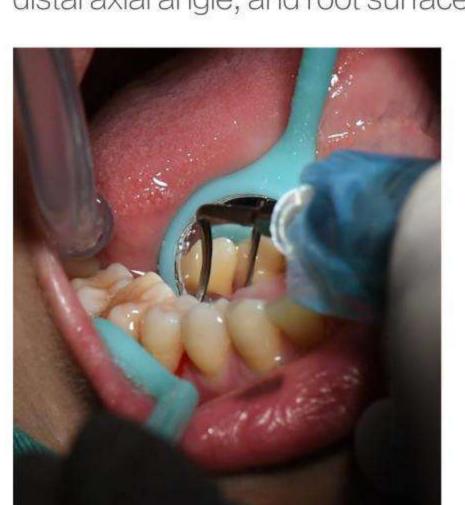



Use P54R for scaling of 14~16,

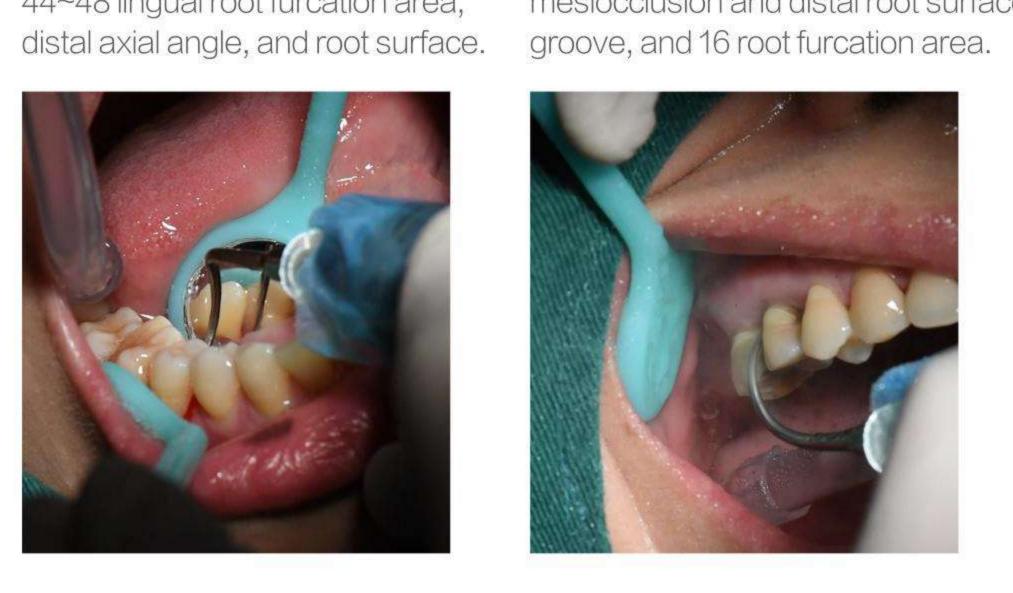


Use P54R for scaling of 44~48 lingual root furcation area,




## Compatible Tips




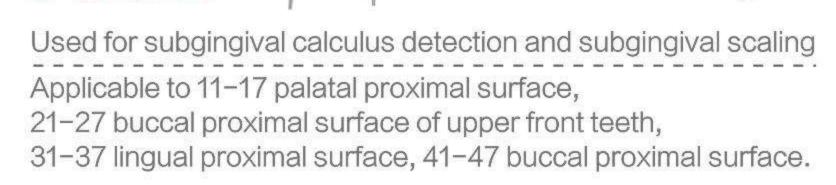





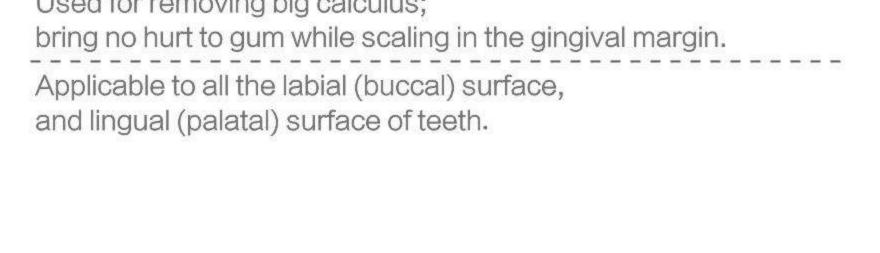

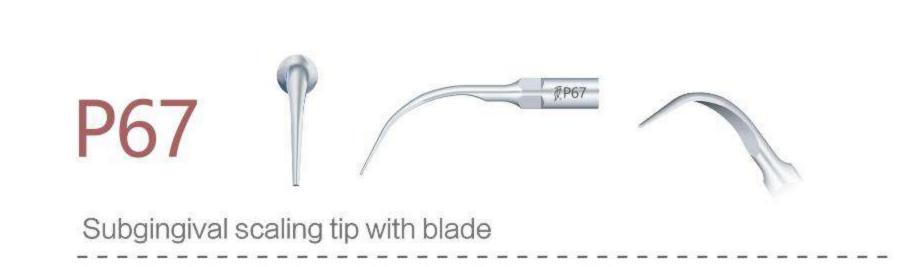


Use the lateral edge at the top 2-3mm of tip.

The direction is parallel to tooth surface.



Europe Sales Dept.: +86-773-5873196, +86-773-2125222 North America, South America & Oceania Sales Dept.:+86-773-5873198, +86-773-2125123 Asia & Africa Sales Dept.:+86-773-5855350, +86-773-2125896 Fax: +86-773-5822450
E-mail:woodpecker@glwoodpecker.com,sales@glwoodpecker.com
Website: http://www.glwoodpecker.com EC REP MedNet GmbH Borkstrasse 10 · 48163 Muenster · Germany

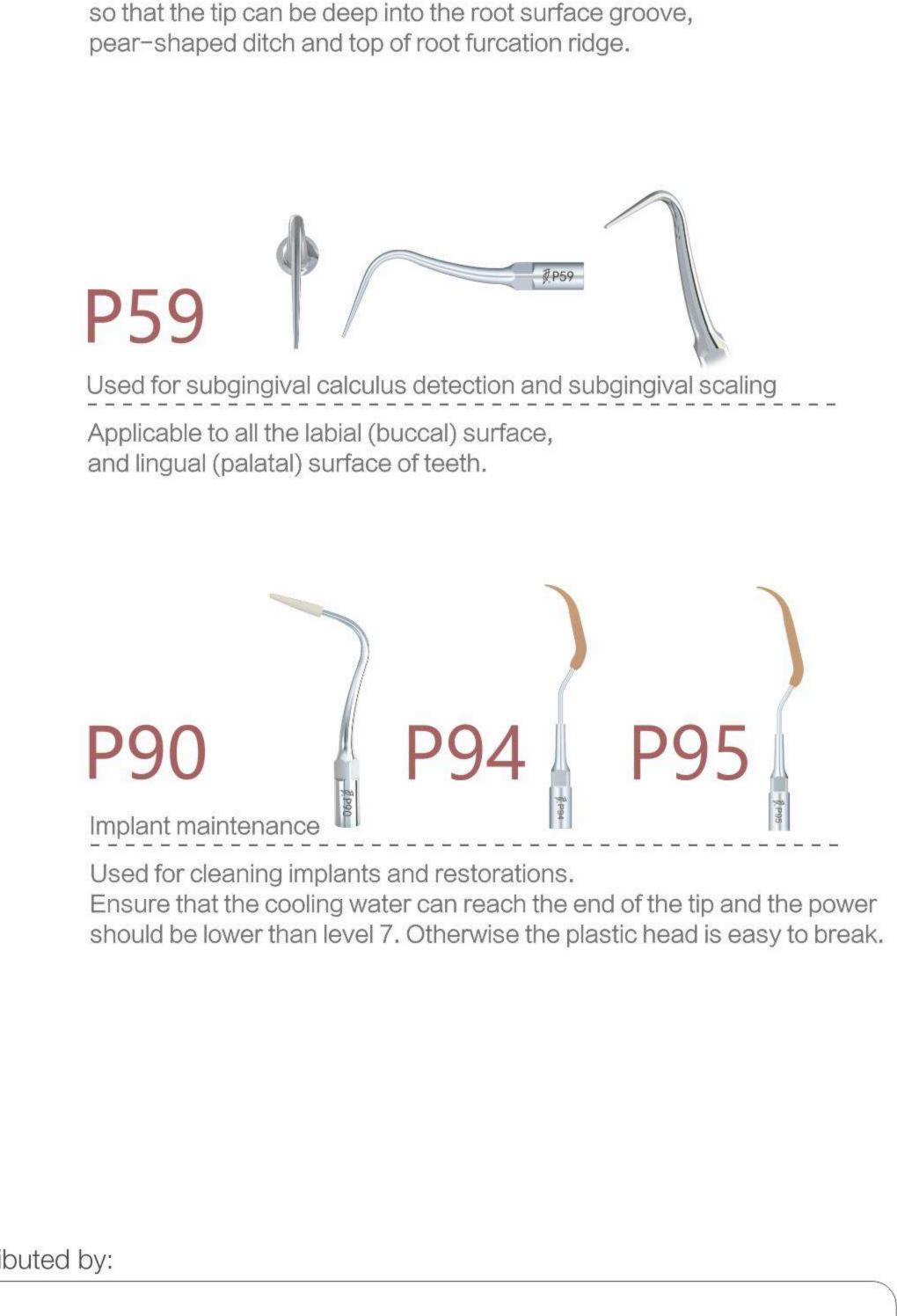

Used for subgingival calculus detection and subgingival scaling
Applicable to 11-17 buccal proximal surface of upper front teeth,

\_\_\_\_\_\_

The curved design can better fit the tooth surface,

21-27 palatal proximal surface, 31-37 buccal proximal surface,

41-47 lingual proximal surface.

Distributed by:

ZMN/WI-10-100

